

Compressed air powered Sprayer for Disinfection product application

Refer to www.dentalsleepbalance.com for demonstration videos.

We recommend contacting your local dental representative to install the equipment to your air system.

1. The device can be adjusted to provide a fine mist that can cover up to 20 feet with fine dispersion to contact surfaces.
2. Please do not point at a person while using a device. Please do not spray in the direction of a person's face.
3. The concentration of disinfection products (especially oxidizing products) can cause premature corrosion of equipment (200 PPM recommended)
4. Some disinfection products can bleach fabrics.
5. Maintain equipment regularly to avoid corrosion of equipment.
6. The system is compatible with dental air supply quick connect hardware.
7. Tube is rated above standards for dental air supply systems.
8. If any oxidation is noted to the sprayer we recommend regular replacement. New options for sprayer will be updated regularly.
9. Powerful oxidation disinfection chemicals can cause oxidation build up in equipment.
10. Do not leave cleaning solution in Sprayer overnight.

Note of disclaimer:

We do not accept returns of products that have been opened or have been used.

Return policy cover: You may return the unopened items for a refund. There will be a 30 percent restocking fee if you choose to return the unopened items.

By making use of any information on this instructional insert, you agree to the following:

NO WARRANTIES: All of the information provided on this instructional insert is provided "AS-IS" and with **NO WARRANTIES**. No express or implied warranties of any type, including for example implied warranties of merchantability or fitness for a particular purpose, are made with respect to the information, or any use of the information, on this site. Dental Sleep Balance and SMG, LLC makes no representations and extends no warranties of any type as to the accuracy or completeness of any information or content on this instructional insert.

DISCLAIMER OF LIABILITY: Dental Sleep Balance, SMG, LLC and their subsidiaries and associated businesses and legal entities specifically **DISCLAIMS LIABILITY FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES** and assumes no responsibility or liability for any loss or

damage suffered by any person as a result of the use or misuse of any of the information or content on any of our websites. Dental Sleep Balance and SMG, LLC assumes or undertakes NO LIABILITY for any loss or damage suffered as a result of the use, misuse or reliance on the information and content on this website.

USE AT YOUR OWN RISK: This equipment is to be used with your clinical judgement only. It is your responsibility to evaluate your own medical and physical condition, or that of your clients, and to independently determine whether to perform, use or adapt any of the information or content or on any written communication or our websites. Any use of this equipment may result in injury. By voluntarily using this equipment, you assume the risk of any resulting injury.

If corrosion occurs replace the sprayer.

We have found several compatible sprayer options based on your needs. Call for pricing. Buying replacement sprayers will be discounted from the full original Complete System purchase.

0.8 Nozzle




1.4 Nozzle



1.4 Nozzle



Full Connection System

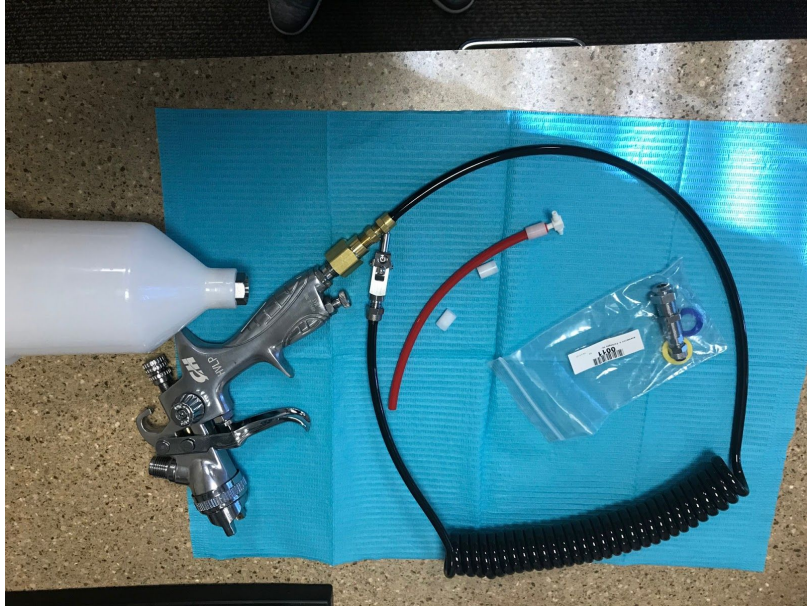

Quick Connect System



Fogger/Sprayer

